





TREPONEMA PALLIDUM REAL TIME PCR

RT AMPLI-SET-TREPONEMA P.

Cat 1.630

The Treponema pallidum is bacteria involved in the syphilis onset, a very common venereal disease. The majority of the Treponema establishes a kind of commensalism with the guest (not being pathogenic), the Treponema pallidum is very virulent. Being very sensitive to the outdoor temperatures the Treponema pallidum bacteria can be transmitted only by direct contact. In addition to the sexual transmission, the Treponema pallidum can infect the fetus via placental during the last period of pregnancy: in this case the disease is called "congenital syphilis". The helical structure of Treponema allows the movements in an highly viscous mean as mucus. Once reached the blood e the lymph, the bacteria infects tissues and mucus. In some cases the Treponema pallidum is transmitted via transfusion of infected blood.

The Kit Treponema pallidum Real-TM is based on two main processes: the extraction Treponema pallidum DNA from samples, amplification using Real-Time PCR and revelation by specific fluorescent probes for Treponema pallidum and Internal Control (IC) used as amplification control for every sample.

Principle of method:) genomic DNA extraction
B) amplification C) revelation by Real-Time PCR instrument
Applicability: on extracted and purified genomic DNA
from whole blood

from whole blood.

Number of Test: 50


Stability: over 12 months if correctly stored.

REAGENTS AND STORAGE

AMPLIFICATION	CONSERVATION
Mix PCR1	-20°C
Mix PCR2	-20°C
Taq Polimerase	-20°C
Sterile water	-20°C
Positive Control	-20°C
Internal Control	-20°C

ANALYSIS OF RESULTS

POSITIVE SAMPLE FOR TREPONEMA P.

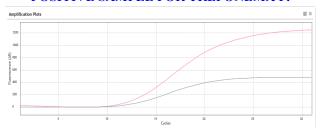

-the signal of the Treponema Pallidum DNA amplification is revealed by FAM channel (pink Plot) -the signal of the internal control amplification is revealed by JOE channel (grey Plot)

Controls	Ct	Ct	Result
	channel	channel	interpretation
	FAM	Joe	•
Internal	Absent	Present	Valid
Control			
Controllo +	Present	Present	Valid
Controllo –	Absent	Absent	Valid
(Acqua Sterile)			

References

Nucleic Acids Res. 24:5021(1996) BR.J Haematol 92:336 (1996) Hemoglobin 19:237 (1995) American J. Of Haeniatol 31:237 (19

American J. Of Haeniatol 31:237 (1989) American J. Of Haeniatol 37:133 (1991)